## GB INSTRUCTION FOR USE

Read the instruction for use before using the product. Store this product at room temperature. Use in well-ventilated areas. External use only.

### 1. CONTENTS

- 1x Instruction for use
- 1x Aerosol dispenser contains 200 ml or 400 ml

### CONTAINS

Butane, Propane, Isobutane, Ethanol, Propan-2-ol, Menthol, Menthyl PCA, Eucalyptus Globulus Oil, Lactamide MEA, Dipropylene Glycol, Limonene, Menthane Carboxamide Ethylpyridine.

### 2. DESCRIPTION

First Aid Cooling provides direct pain relief for acute sports injuries, bruises and sprains. The cold spray provides intense cooling and minimises emerging swellings and bruises.

### 3. INTENDED USE / INDICATIONS FOR USE

This product is indicated for the treatment of minor acute sports injuries.

The product is intended for use on:

- healthy people over 12 years of age
- only on intact skin
- external use only

### 4. INSTRUCTIONS FOR USE

1. Shake well before use.
2. Keep the aerosol dispenser 15 cm away from the skin.
3. Spray 5 second max. in a circular motion on the skin.

Stop applying the product immediately if the skin colors white or white frost appears on the skin. Stop using the product immediately when skin irritation occurs.

Use in well-ventilated areas.

### 5. WARNINGS & PRECAUTIONS

Read label before use. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Avoid breathing dust / fume / gas / mist / vapours / spray. Use only outdoors or in a well-ventilated area. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice / attention. If skin irritation or rash occurs: Get medical advice / attention. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. Extremely flammable aerosol. Pressurised container: May burst if heated. Causes serious eye irritation. May cause an allergic skin reaction. Store in a well-ventilated place. Protect from sunlight. Do not expose to temperatures exceeding 50 °C / 122 °F. Dispose of contents / container in accordance with local / regional / national / international regulations.

### 6. CONTRAINDICATIONS

DO NOT USE

- if sensitive to one of the ingredients
- for diabetics and people with circulatory problems
- when the skin condition is poor, irritated or inflamed
- near the face, nose, eyes, mucous membrane or genitals

When pregnant or breastfeeding consult a doctor before use.

### 7. POTENTIAL ADVERSE EVENTS

Stop applying the product immediately if the skin colors white or white frost appears on the skin.

### 8. COMPLAINTS & CUSTOMER SERVICE SUPPORT

Contact the manufacturer with questions / complaints or to report an adverse reaction.

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