## GB INSTRUCTION FOR USE

Read the instruction for use before using the product. Store at room temperature. Avoid storing product in direct sunlight and/or high temperatures. External use only.

### 1. CONTENTS

- 1x Instruction for use
- 1x Aerosol dispenser contains 200 ml

### CONTAINS

Hexamethyldisiloxane, Propan-2-ol, Butane, Ethanol, Propane, Isobutane.

### INGREDIENTS ACCORDING TO DETERGENTS GUIDLINE 648/2004/EC

Aliphatic hydrocarbons ≥30%.

### 2. DESCRIPTION

Tape Remover is an aid to easily remove sports and kinesio tapes. The spray dissolves the adhesive layer of the tape quickly and effectively making removal less painful. Developed for (top) athletes, sports carers, masseurs and physiotherapists.

### 3. INTENDED USE / INDICATIONS FOR USE

This product is indicated for the removal of sports tape and other products using adhesives on the skin.

The product is intended for use on:

- healthy people over 12 years of age
- external use only

**Do not** spray directly on the skin.

### 4. INSTRUCTIONS FOR USE

1. Shake well before use.
2. Spray the product, 5-7 seconds from a distance of 20-30 cm, in a circular motion on the adhesive tape.
3. Allow the product to soak the tape.
4. Remove the tape gently
  - if the tape pulls too much on the skin; re-apply.

### 5. WARNINGS & PRECAUTIONS

**DANGER** Extremely flammable aerosol. Pressurised container: May burst if heated. Causes serious eye irritation. May cause drowsiness or dizziness. Very toxic to aquatic life. Toxic to aquatic life with long lasting effects. Buildup of explosive mixtures possible without sufficient ventilation.

Read label before use. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Avoid breathing dust/fume/gas/mist/vapours/spray. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Store in a well-ventilated place. Avoid release to the environment. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. Dispose of contents/container in accordance with local/regional/national/international regulations.

### 6. CONTRAINDICATIONS

DO NOT USE

- if sensitive to one of the ingredients
- directly on the skin or on wounds
- in or around facial areas or genitals

When pregnant or breastfeeding consult a doctor before use.

### 7. POTENTIAL ADVERSE EVENTS

May cause drowsiness or dizziness, consult your physician immediately.

### 8. COMPLAINTS & CUSTOMER SERVICE SUPPORT

Contact the manufacturer with questions/complaints or to report an adverse reaction.

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