GB INSTRUCTION FOR USE

Read the instruction for use before using the product. Store at room temperature. Avoid storing product in direct sunlight and/or high temperatures. Use in well-ventilated areas. External use only.

1. CONTENTS

1x Instruction for use
1x Aerosol dispenser contains 200 ml

INGREDIENTS

DIMETHICONE, BUTANE, PROPANE, DIETHYLHEXYL CARBONATE, ISOBUTANE

2. DESCRIPTION

Zero Friction creates a thin, smooth protective layer on the skin, to prevent and reduce friction or abrasion. The spray must be applied before or during sports effort. Athletes experience more comfort during training and competitions.

FEATURES

- prevents and reduces friction and abrasion
- reduces the risk of blistering and/or skin irritation
- preventive protection product
- provides dry, chapped or rough skin

The spray will help you to put on and take off wet-suits or diving suits, as a result of the smooth layer. It also prevents hot-spots caused by sports shoes or clothing, compression clothing and/or (sporting) aids. Vital piece of kit for walkers, runners, triathletes, mountain climbers, watersport enthusiasts, cyclists and/or other endurance sports people.

3. INTENDED USE / INDICATIONS FOR USE

The spray prevents and reduces friction, abrasion, blistering and/or skin irritation. The product is intended for use on:

- healthy people over 12 years of age
- the skin as a protective layer
- preventive against blisters
- only on intact skin
- external use only

4. INSTRUCTIONS FOR USE

Can be applied without smearing.

1. Use in a well-ventilated area only.
2. Shake well before use.
3. Keep the aerosol dispenser 10 cm away from the skin.
4. Apply the spray evenly.
5. Repeat until the desired coverage has been reached.

May become slippery after application. Risk of slipping.

Stop using the product immediately when skin irritation occurs.

5. WARNINGS & PRECAUTIONS

Read label before use. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat; hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Extremely flammable aerosol. Pressurised container: May burst if heated. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122°F. If skin irritation occurs: Get medical advice/attention.

6. CONTRAINDICATIONS

DO NOT USE

- if sensitive to one of the ingredients
- if skin irritation occurs
- when the skin is irritated or inflamed
- near the mouth, nose, eyes or genitals

When pregnant or breastfeeding consult a doctor before use.

7. POTENTIAL ADVERSE EVENTS

Stop using the product immediately if skin irritation occurs.

8. COMPLAINTS & CUSTOMER SERVICE SUPPORT

Contact the manufacturer with questions/complaints or to report an adverse reaction.

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